

90XL Displays Being Exposed to Excessive Fluid

During our on-going evaluation of Service Returned 90XL Display Modules it has been determined that in some cases the Display Module is being exposed to excessive fluid. The Display module's keypad gasket is designed to prevent moisture from reaching the electronics. However, excessive exposure of the Display Module to liquids can result in damage to the internal electronics. Wiping down the 90 XL Display Module with a damp cloth is recommended and acceptable. The cloth should not be dripping any amount of fluid. Please avoid exposing the 90XL Display Module to excessive fluids either directly or with an over saturated cloth. Please refer to page 35 of the 90XL User Manual for more information on cleaning the Display Module located here:
http://www.mesalabs.com/wp-content/uploads/2012/02/90XLManual.901120027.RevF_.20110208.pdf

Thank you.